

# Full Body Sling & Full Body Sling Net



Divided Legs  
Reclined Seating  
Head Support



The Full Body Sling is a comfortable sling without reinforcements, handles or padding. It is available in either Polyester or Polyester Net and offers good support for the entire body. The Full Body Sling can be used for most lifting situations and fits most patients.

## Versatile Design

The Full Body slings are designed to create a reclined seated position which is extremely helpful if the patient has poor trunk stability and/or a tendency to overstress themselves (spastic performance) or are hypertonic. The reclined position prevents the user from sliding out of the sling. The reinforced head support makes this a suitable sling for lifting patients from a flat surface such as a floor.

## Two Fabric Choices

The Full Body slings are available in either Polyester or Polyester Net fabrics. Both can be left behind the patient for a short period of time after the transfer is complete. The low friction Polyester fabric is easy to apply

and remove and the Polyester Net is ideal for most bathing situations.

## Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced loops and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180.\*



### Leg Padding and Supports

Ideal for users with sensitive skin or pain. Part no.: 21146-S, M, L Reinforced supports. One size. Part no. 25102



### Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



### Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 [2- or 3-Point - Quick Release], 55619 [4-Point - Attach to leg hook]

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Use to/from the floor


Suitable for transfers from the floor.



### Use for showering


Only Polyester Net fabric can be used for this application.

## Care and Features

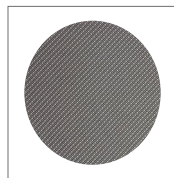


### Polyester

Durable, low-friction, easy to clean, apply and remove.

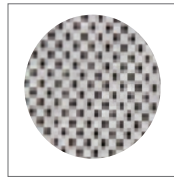

### Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering.

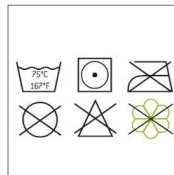

### Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.



## Technical Specifications

Part Number	Size	Colour		Model #25005			Model #25010			
				A	B	C	A	B	C	
<b>Polyester Net</b>	<b>Polyester</b>									
25005-JM	n/a	Junior Medium	JM	22cm / 8.7"	71cm / 28"	80 cm / 31.5"	-	-	-	
25005-S	25010-S	Small	S	26cm / 10.2"	76cm / 29.9"	86cm / 33.9"	31cm / 12.2"	85cm / 33.5"	90cm / 35.4"	
25005-M	25010-M	Medium	M	26cm / 10.2"	85cm / 33.5"	95cm / 37.4"	40cm / 15.8"	92cm / 36.2"	91cm / 35.8"	
25005-L	25010-L	Large	L	34cm / 13.4"	96cm / 37.8"	95cm / 37.4"	40cm / 15.8"	98cm / 38.6"	100cm / 39.4"	
25005-XL	25010-XL	Extra Large	XL	36cm / 14.2"	105cm / 41.3"	106cm / 41.7"	44 cm / 17.3"	105cm / 41.3"	102cm / 40.2"	
n/a	25010-XXL	Double Extra Large	XXL	-	-	-	44 cm / 17.3"	111cm / 43.7"	106cm / 41.7"	
n/a	25010-XXXL	Triple Extra Large	XXXL	-	-	-	44 cm / 17.3"	118cm / 46.5"	108cm / 42.5"	
Maximum User Weight			300kg/661lb							

\* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]