




HUMAN CARE

Your Life. Your Way.

Fitting Protocol

Silhouette Sling Net & Soft
Models 25015, 25019

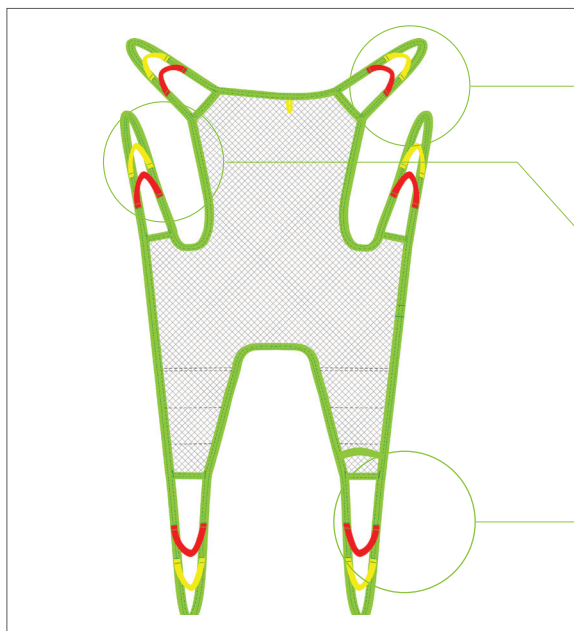

- ☐ Green Strap
- ☐ Yellow Strap
- ☐ Red Strap

Extension Loops

- ☐ 21180-10 (10cm/3.9")
- ☐ 21180-15 (15cm/5.9")
- ☐ 21180-20 (20cm/7.8")
- ☐ 21180-25 (25cm/9.8")
- ☐ 21180-35 (35cm/13.7")

- ☐ Green Strap
- ☐ Yellow Strap
- ☐ Red Strap

Extension Loops

- ☐ 21180-10 (10cm/3.9")
- ☐ 21180-15 (15cm/5.9")
- ☐ 21180-20 (20cm/7.8")
- ☐ 21180-25 (25cm/9.8")
- ☐ 21180-35 (35cm/13.7")

- ☐ Green Strap
- ☐ Yellow Strap
- ☐ Red Strap

Extension Loops

- ☐ 21180-10 (10cm/3.9")
- ☐ 21180-15 (15cm/5.9")
- ☐ 21180-20 (20cm/7.8")
- ☐ 21180-25 (25cm/9.8")
- ☐ 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 300kg/660lb

- ☐ **Model 25015 - Polyester Net**
- ☐ **Model 25019 - Soft Air Mesh**

Transfer Situation:




Available sizes:


☐ Indicate size required



Leg Placement:

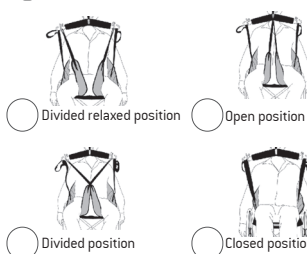

Suspension:

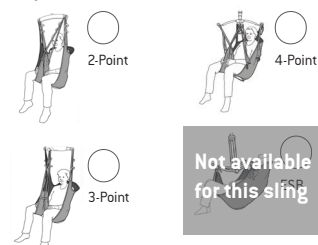

Accessories required:



- ☐ 21146 Leg Padding - Small
- ☐ 21146 Leg Padding - Medium
- ☐ 21146 Leg Padding - Large



- ☐ 25102 Reinforced Leg Support - One Size



- ☐ 50866 Upholstered Cover for Hanger Bar
Do not use in wet environments.

Patient Information:

Name

Facility

Lift Used

Room

Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Silhouette Sling is a comfortable sling without reinforcements, handles or padding. It adapts to the patient's body to offer good support. It can be used in most lifting situations and its narrow design makes it an optimal choice for patients who are smaller in stature. The Silhouette Sling is available in Polyester Net and Soft Air Mesh fabric, both of which can be left behind the patient for a short period of time after the transfer is complete.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name	
Date	Position/Title



Important! This fitting protocol does not replace the user manual. Read all instructions before use!