




Fitting Protocol

Positioning Sling
Model NSB-401-ST, NSB-410-ST,
NSB-451-U

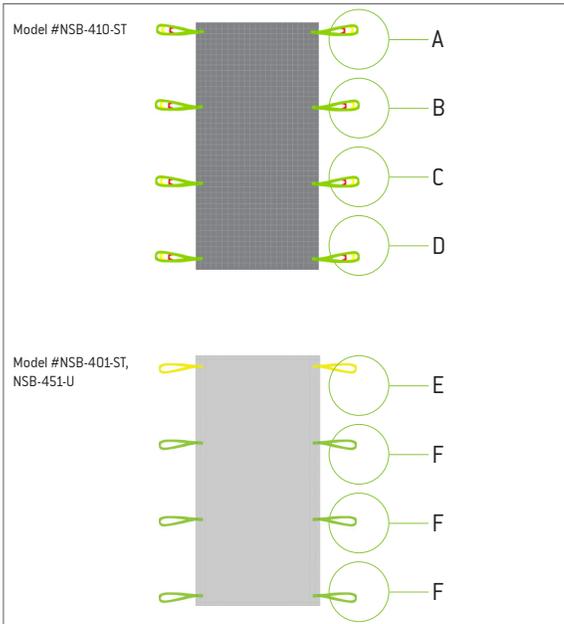

Strap A

Red Strap
 Yellow Strap
 Green Strap

Strap B

Red Strap
 Yellow Strap
 Green Strap

Strap C

Red Strap
 Yellow Strap
 Green Strap

Strap D

Red Strap
 Yellow Strap
 Green Strap

Yellow Strap E
 Green Strap F

Extension Loop Size:
Strap A: _____
Strap B: _____
Strap C: _____
Strap D: _____

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.




Red Strap
Upright Position
Least hip angle

Yellow Strap
Semi-Recumbant
Mid-hip angle

Green Strap
Increased Recline
Less hip flex and greatest angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

For Models NSB-401-ST and NSB-451-U angle adjustment not applicable.

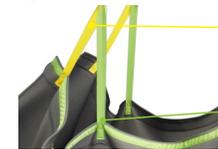

Yellow Strap
Head positioning

Green Strap
Body positioning

Maximum User Weight: 454kg/1000lb

Maximum User Weight: 318kg/700lb

- Model NSB-401-ST, NSB-410-ST, Ventilated Polyester
- Model NSB-451-U - Single Patient Use Polyester/Cotton

Transfer Situation:



Available sizes:

Indicate size required


ONE SIZE **L** **XL**

Model #NSB-410-ST only

Leg Placement:



Suspension:

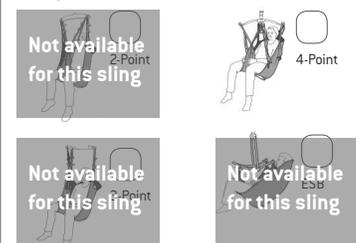

Accessories required:



- Extension Loops
- 21180-10 (10cm/3.9")
 21180-15 (15cm/5.9")
 21180-20 (20cm/7.8")
 21180-25 (25cm/9.8")
 21180-35 (35cm/13.7")

Patient Information:

Name _____

Facility _____ Room _____

Lift Used _____ Hanger Bar Used _____

HUMAN CARE SWEDEN (HQ)
Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE AUSTRALIA
Unit 4, 49 Bertie Street
Port Melbourne, VIC 3207
Phone : +61 1300 132 736
Fax : + 61 (03) 9645 4954
info.au@humancaregroup.com

HUMAN CARE CANADA
10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND
Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES
8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Positioning Sling is designed to assist a caregiver to safely and securely perform safe patient handling during sudden or intended positioning/repositioning, turning and lateral transfer from a supine position. This sling is offered in a reusable or single patient use model. The reusable model can be placed behind the patient much like a sheet and can remain behind the patient for short periods of time. The unique material facilitates air flow and breathability and does not promote skin breakdown. The single patient use model offers the convenient option of disposal after use combined with the added benefit of hygiene control. All slings fit standard hospital beds.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name	
Date	Position/Title



Important! This fitting protocol does not replace the user manual. Read all instructions before use!

Additional Documentation

Can be found on our website at www.humancaregroup.com/product-category/slings



User Manual



Sling Care Instructions



Sling Inspection Form

