




HUMAN CARE

Your Life. Your Way.

Fitting Protocol

Multi Sling
Model 25105

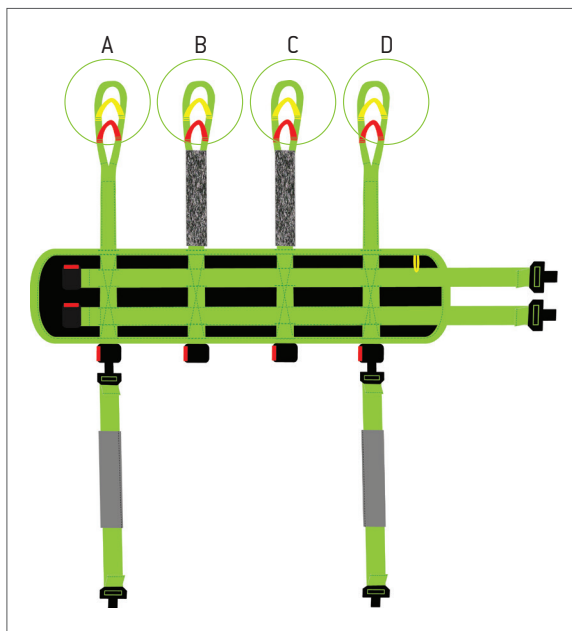

Strap A (Front)

- ☐ Red Strap
- ☐ Yellow Strap
- ☐ Green Strap

Strap B (Back)

- ☐ Red Strap
- ☐ Yellow Strap
- ☐ Green Strap

Strap C (Back)

- ☐ Red Strap
- ☐ Yellow Strap
- ☐ Green Strap

Strap D (Front)

- ☐ Red Strap
- ☐ Yellow Strap
- ☐ Green Strap

Components of Multi Sling:



Chest Belt
Part no. 25106




Groin Support
Part no. 25108




Shoulder Support
Part no. 25109

Choose the loop combination that will give the best height for the intended use. Match colours on both sides of the patient (2 front straps and 2 back straps of chest belt).

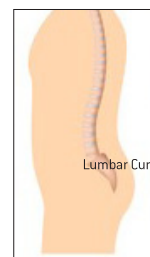

When fitting the chest belt portion of the Multi Sling, ensure that it rests on the lumbar curve above the coccyx.

If applying a waist belt, insert through the inner loops of the chest belt.

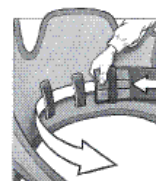

The Multi Sling should be applied while the patient is in the seated position.

Maximum User Weight: 150kg/330.7lb

☐ Model 25105 - Polyester

Transfer Situation:



Available sizes:

☐ Indicate size required




Leg Placement:




Divided relaxed position



Open position

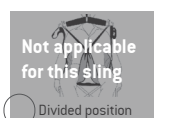

Divided position

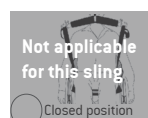

Closed position

Suspension:




2-Point



4-Point



3-Point

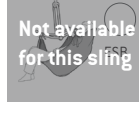

Accessories required:



- ☐ 50866 Upholstered Cover for Hanger Bar. **Do not use in wet environments.**



- ☐ 25150 Neck Cushion



- ☐ 25145 Waist Belt - Velcro
(Inserted through inside loops of chest belt)



- ☐ 25146 Waist Belt - Safety Lock
(Inserted through inside loops of chest belt)



- ☐ 25107 Hip Belt

Waist Belt (Select Size)

Available Sizes:
-60 (60cm/23.6")
-70 (70cm/27.6")
-80 (80cm/31.5")
-90 (90cm/34.4")
-100* (100cm/39.4")
-110 (110cm/43.3")
-120 (120cm/47.3")
-130 (130cm/51.2")
*Model 25146 only

Patient Information:

Name

Facility

Lift Used

Room

Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Multi Sling is designed for gait training and rehabilitation exercises. It can be used in many situations and together with a wide range of accessories can act as support during standing, walking and balancing exercises as well as for visits to the toilet or dressing and undressing. The Multi Sling is suitable for patients with some body stability and partial use of their legs and is approved for 2-point, 3-point or 4-point suspension. When used with a hip belt, it provides excellent support for patients with serious functional impairments such as stroke. Adding a groin support provides additional support and safety.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name	
Date	Position/Title



Important! This fitting protocol does not replace the user manual. Read all instructions before use!