


# General Purpose Sling

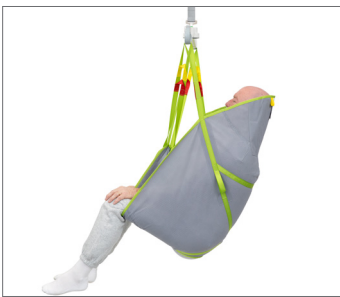

Single patient use  
or Reusable models  
Loop or Clip



Our General Purpose slings have a high weight capacity and a generous overall width to offer a stretched seating position that is well-suited for lifting heavier users in all situations. The integrated head support allows the patient to be lifted even from a floor. The sling is available in SPU and Reusable, with clips and with loops.

## Two Convenient Options

The General Purpose sling is available with either reinforced loops that ensure durability and ease of use, or with clips for easy and secure attachment. Either option is an ideal solution for sudden positioning change requirements. These slings are available in adult sizes Small to Extra Large and can carry a maximum capacity of 318kg/700lb.

## Promotes Infection Control

The single patient use (SPU) slings offer the convenient option of disposal after use combined with the added benefit of hygiene control. Since the sling can only be used by one patient at a time, facility infection control protocols are easily supported.

## Optimal Features

These slings feature divided legs and a high back for added support. They are made of a strong but soft polyester/cotton blend and are ideal for a seated, horizontal or semi-recumbent transfer. The unique material wicks away fluids and can be left behind a patient. Model #NSB-104 can be used with a spreader bar. Model #NSB-105 features loops that can be attached to a hanger bar with 2-point, 3-point or 4-point suspension.



Walking Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible Chairs

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.: 21180-\*



### Leg Padding

Soft padded supports for legs. Ideal for users with sensitive skin or pain. Synthetic sheepskin. Part no.: 21146-S, M, L



### Hanger Bar

For NSB-105 (loop sling). For 2-, 3-, or 4-point suspension.



### Spreader Bar

Powered Spreader Bar. For NSB-103C and NSB-104C (clip slings).

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Use to/from the floor

Suitable for transfers from the floor.



### Single Patient Use

Sling is intended for use by a single patient at a time. Dispose after use.

## Care and Features




### Clip or Loop Sling

Two convenient attachment options - clips or loop slings.



### Reinforced loops

Colour coded reinforced loops for easy determination of required positioning.

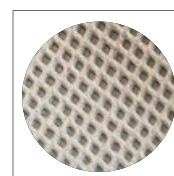


### Non-woven polypropylene

Material allows fluids to wick away from patient. Do not launder. Single Use.



## Technical Specifications

Article number		Size	Color	A	B	C
NSB-103C-S	General purpose sling with clips	Small	S	30cm / 12"	84cm / 33"	78cm / 31"
NSB-104C-S	General purpose sling with clips SPU					
NSB-103C-M	General purpose sling with clips	Medium	M	43cm / 17"	88cm / 35"	97cm / 38"
NSB-104C-M	General purpose sling with clips SPU					
NSB-103C-SL	General purpose sling with clips	Large	L	53cm / 21"	103cm / 41"	114cm / 45"
NSB-104C-L	General purpose sling with clips SPU					
NSB-103C-XL	General purpose sling with clips	Extra Large	XL	68cm / 27"	114cm / 45"	133cm / 53"
NSB-104C-XL	General purpose sling with clips SPU					
NSB-105-S	General purpose sling loops SPU	Small	S	30cm / 12"	84cm / 33"	78cm / 31"
NSB-105-M	General purpose sling loops SPU	Medium	M	43cm / 17"	88cm / 35"	97cm / 38"
NSB-105-L	General purpose sling loops SPU	Large	L	53cm / 21"	103cm / 41"	114cm / 45"
NSB-105-XL	General purpose sling loops SPU	Extra Large	XL	68cm / 27"	114cm / 45"	133cm / 53"
Maximum user weight: 318kg/700lb						

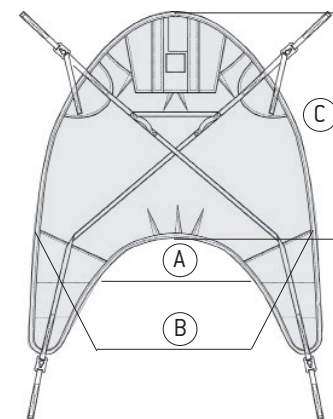

The drawing illustration shows NSB-104C General purpose sling with clips - SPU

[www.humancaregroup.com](http://www.humancaregroup.com)

### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
+46 8 665 35 00  
info@humancaregroup.com

### HUMAN CARE CANADA

10-155 Colonnade Road  
ON K2E 7K1 Ottawa  
+1 613 723 6734  
info.ca@humancaregroup.com

### HUMAN CARE THE NETHERLANDS

Elsperweg 124  
8076 PA Vierhouten  
+31 577 41 21 71  
info.nl@humancaregroup.com

### HUMAN CARE UNITED STATES

8006 Cameron rd, Suite K  
TX 78754 Austin  
+1 512 76 7199  
info.us@humancaregroup.com