# HUMAN CARE

Your Life. Your Way.

Dolphin II Commode, Shower chair /  
Dusch-, toalettstol / Dusch-, WC-Stuhl /  
Douche, Toiletstoel 90561

Service manual / Servicemanual / Servicehandbuch /  
Servicehandleiding

## Content

### ENGLISH

Service Manual	3
Compliances and Standards	4
Useful information	4
Service manual description	5
Tools & Maintenance	5
Troubleshooting & Solving	7
Spare parts list	13

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# Service Manual

## Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted.  
Before use, always check that the product  
is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

## Compliances and Standards

---

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 17966:2016.

## Useful information

---

### a) Service manual

The service manual is created for technicians involved in servicing of Dolphin II commode/shower chair. Safe performance of product service procedures requires experience, technical skills and general knowledge of safety rules.

This manual uses warnings and cautions to pay special attention to service procedures.

**Warning** – this information tells us about the possibility of safety risk of technicians and user.

**Caution** – this information tells us about actions that can cause potential damage of the product.

### b) Dolphin II commode/shower chair installation and operation

For installation and operation of the product, see the user manual.

### c) Service of Dolphin II chair

The service can be only made by Human Care's authorized service team or authorized units by Human Care.

Human Care is not responsible for the consequences of repairs or modifications carried out by unauthorized personnel or by use a non-original spare parts.

### d) Service documentation

Service documentation and service repair document are available on a website accessible for authorized service.

### e) Safety precautions

The Dolphin II chair design provides maximum of safety. Safety of the Dolphin chair are ensured by safety standards IEC60601-1.

Read these cautions carefully before start servicing!

**Note** - do not perform service repairs and functional testing when the patient is on the device.

### f) Additional information

In case of any problems, please contact the intermediary or manufacturer's service.

#### Service contact details:

HUMAN CARE SWEDEN (HQ)

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117 43 Stockholm

Phone: +46 8 665 35 00

Fax: +46 8 665 35 10

Email: [info.se@humancaregroup.com](mailto:info.se@humancaregroup.com)

## Service manual description

The service manual contains information about repairing, maintaining and identification of chair. The device can be identified by using the

identification code which is placed on the name plate.

## Tools & Maintenance

**Note** – if the plug from battery charger does not match a socket in your country, you can use an adapter.

### Tools

Standard set of tools is required for servicing of the product:

Allen key set

Ordinary set of various keys

Flat screwdriver

Hammer

Volt meter

# Dolphin II Service Manual

## Device maintaining

All movable joints must be lubricated every six months or when experiencing squeaks or other noise. These nodes are:

- axle transport wheels,
- axle actuator mounting,
- jib arm bearing sleeve.

We recommend using any available penetrating lubricants.

Report all loose connections that are supposed to be fixed to the manufacturer's authorized service agent and stop using the product until the problem is resolved.

This device is equipped with electric actuator which allows to adjust the height of the seat. In order to ensure proper performance of the actuators check the actuators for unusual sounds every 3 months. These unusual sounds contain rattles, squeaks, creaks. Any perceived performance deficiency must be reported to Human Care technical service immediately ceasing device operation until remove their causes.

Not less frequently than every 1 years and after each failure/repair of actuator technical service must check the product for electric safety.

The minimal range of the review should include:

- check if there were no mechanical wiring damages of the device,
- check if there were no mechanical damages of the controller and the actuators.

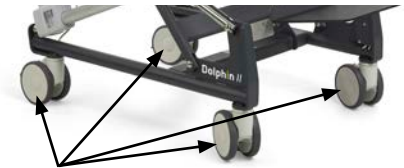
See checklist in user manual.

## Basic periodic review

One year after production date or every 8000 cycles of using the product, the indicator lamp light up.

If this LED is on, operator must check condition of the lift (cables connections, looseness on the frame etc.).

Press and hold "up" and "down" button for 5 sec., to turn off the light after review.

Transport wheels

Actuator mounting

Jib arm bearing sleeve

LED indicator

# Troubleshooting & Solving

## Troubleshooting

Symptom	Possible cause - solution
Device doesn't work properly.	Diagnose the product.
Height adjustment doesn't work.	Replace main actuator.
Tilt adjustment doesn't work.	Replace secondary actuator.
Problem with moving device.	Check/replace castors.

## Diagnose the product

In case of problems with any adjustments, please complete following steps:

### Step 1:

- Check the battery level on the hand control. If any led is lighting, please check the battery with the voltmeter,
- If the battery is empty, please charge it,
- If after a few hours the battery is still empty, please check if the charger is working correctly by using the voltmeter. If the charger is ok, you have to replace the battery for a new one.

Checking voltage on battery

Checking voltage on charger

### Step 2:

If the battery level is correct, check if the wires are connected properly to the control box in the bottom part of the device.

Checking wires connection

# Dolphin II Service Manual

## Step 3:

If everything looks good so far, but the device still doesn't work, for further diagnose, spare parts will be necessary (hand control, control box and actuator).

- At first, replace the hand control for new one (or switch with another device if it's possible) and check,
- Next, prepare a new control box and connect it with slots from device,
- Now test what is wrong, control box, hand control or actuator.

**Caution** – in case of situation that one of actuators is not working, make sure that the problem is with the actuator, by swapping the wires 1 and 2 for actuators in the control box and use the hand control to test (a working actuator means a faulty control box).

## Step 4:

Replacing of control box starts with taking off the metal cover to get better access to two main screws under the battery. Then unscrew these screws and take of the control box with metal holder.

Removing two screws from both sides

Controller without the battery and metal cover

Removing two main screws

Removing along with the holder

## Replace main actuator

**Note** – These procedure requires to use a brand new 3M Scotchlock connectors (waterproof).

**Caution** - Following procedures require turn down device on aside. To protect it from any damage or scratches we strongly recommend to prepare some kind of protection (card boards, sponges, etc.) on the floor.

### Step 1:

Use a flat screwdriver to remove the star lock cap and circlips from the upper rod holding the actuator (from only one side of the pin).

**Caution** – To prevent device from any damages or scratches use some cloth or piece of cardboard to put between tool and frame.

Removing starlock cap

Example how to secure frame from scratch

Removing circlip 1

# Dolphin II Service Manual

## Step 2:

Remove the pin.

Taking out the pin

Actuator without the pin

## Step 3:

Do the same operation for bottom connection actuator with frame.

Taking out the pin

## Step 4:

Cut the cable tie and gently pull the wire out of the frame until the cable connectors appears. Then cut the wires, remove old actuator and use a new scotchlock connectors to connect the new one.

3M scotchlok connector

Cutting the cable tie

Connections outside of the frame

**Step 5:**

Install the new actuator in the reverse order to dismantling, carefully tighten the gland.

**Replace secondary actuators****Step 1:**

Use a flat screwdriver to remove the star lock cap and circlips from one of the rods holding the actuator (only one side of the pin). Do the same from the other side.

Removing star lock cap

Circlip removing

Taking out the pin

Circlip removing

Rubber washer 1

**Caution** – Pay special attention to not lose rubber washers.

# Dolphin II Service Manual

## Step 3:

Use a small screwdriver to disconnect actuator's plug from socket.

Removing the safety lock step

Removing the safety lock step

Plug and socket disconnected

Whole actuator disconnected

## Step 4:

Install the new actuator in the reverse order to dismantling, carefully tighten the gland.

**Caution** - Process of replacing of the other, secondary actuator is the same.

## Wheel replacement

Use the 19" socked wrench to unscrew the nut inside whole mechanism.

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