PM-2078

PM-2076

PM-2070

## Rollstuhlslide Sliding Mat Wheelchair

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Bruksanvisning · Käsikirja

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### **1 Allgemeine Informationen**

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## Table of Contents

### **1 General Information**

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

### **2 Safety information**

- 2.1 Symbols used
- 2.2 Function check

### **3 Using the product**

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

### **4 Product marking / CE**

### **5 Cleaning, care and disinfection**

### **6 Technical Data**

### **7 Warranty**

### **8 Disposal information**

# 1 General Information

Thank you for choosing a Petermann product.

## **Please read the user manual carefully before using the product for the first time!**

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Sliding Mat Wheelchair is part of a product line which is used specifically to aid in raising patients who have slid forward in the wheelchair in home care settings, hospitals and other health care facilities.

The product may only be used for the specified purpose and under the specified conditions.

The Sliding Mats Wheelchair are a safe transfer aid, which allows for a setup transfer which is pleasant and comfortable for the patient as well as being simple and back-friendly for the users.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

## 1.1 Product versions

**Item no.: PM-2070** Sliding Mat Wheelchair

**Item no.: PM-2076** Sliding Mat Wheelchair with loops

**Item no.: PM-2078** Sliding Mat Wheelchair with loops

## 1.2 Accessories

No accessories are available for this device.

## 1.3 Scope of delivery

Sliding Mat Wheelchair

User manual

## 1.4 Storage, transport

Store/transport                      at -5 C – 40 C  
Other:                                      protect from sunlight

## 1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

# 2 Safety information

**WARNING indicates a potential danger which could lead to personal injury or material damage.**

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- In case of direct skin contact, observation of changes in skin appearance is indicated.
- Weight limits must be observed (200 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the product properties.

## 2.1 Symbols used

	60°C laundry		max. load capacity see item 6
	Do not bleach		Manufacturer
	Dry at low temperatures		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

## 2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

 <p><b>Visual inspection</b> Ensure that material, seams, handle loops and buckles are undamaged and in functional order.</p>	 <p><b>Anti-slip function</b> No anti-slip function: product can be moved Good anti-slip function: Product cannot be moved</p>
 <p><b>Sliding properties</b> Place the product on a firm surface Pull your hands over the product with pressure Good glide - easy gliding Low glide: no gliding/moving possible</p>	
 <p>➤ <b>Defective products may no longer be used!</b></p>	

## 3 Using the product

### 3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Sliding Mats Wheelchair are suitable for all applications involving people who require assistance. Thanks to the reduced friction, the capabilities of the patient can be put to use when Sliding Mats Wheelchair are used. The effort required on the part of the user is reduced.

- Reduction of shear and friction forces at the sacrum
- Reduction of shear and friction forces in the area of the ischial tuberosity

**This product may be reused.**

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

## 3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Agitated patient
- Elevated extensor tone

## 3.3 Application risks

- The patient's level of activity was assessed incorrectly → risk of falling
- Improper placement → risk of falling

## 3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

## 3.5 Application illustration

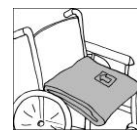
## 3.6 Application

The Sliding Mat Wheelchair was developed for patients who have slid forward in the wheelchair. The Sliding Mat Wheelchair can be used to return patients to a stable sitting position.

All Sliding Mats Wheelchair only slide in one direction and are placed underneath the patient permanently. The sewn-on arrow indicates the direction of movement.

### 3.6.1 PM-2070 Sliding Mat Wheelchair 500 x 540 mm (tubular sliding mat)

The Sliding Mat Wheelchair is a tubular sliding mat. The arrow indicates the direction of movement. Depending on the circumstances, the wheelchair user can place themselves in a stable position on their own or with assistance. With smooth wheelchair seats, an anti-slip mat must be placed underneath the Sliding Mat Wheelchair.

### 3.6.2 PM-2076 Sliding Mat Wheelchair 1300 x 450 mm (open sliding mat)

The Sliding Mat Wheelchair is an open sliding mat. The arrow indicates the direction of movement.

#### Preparation

- Use the straps to attach the Sliding Mat Wheelchair to the wheelchair backrest at the back of the seat. **Not on the sides!**
- Fold the Sliding Mat Wheelchair and place it on the seat.
- The sliding surfaces must face in towards one another.
- The first pulling loops are positioned against the sides of the backrest.

## Transfer

- The wheelchair user can be placed in a stable position with assistance by pulling on the loops.
- This process can be repeated several times.

### 3.6.3 PM-2078 Sliding Mat Wheelchair 1350 x 430 mm (open sliding mat)

The Sliding Mat Wheelchair is an open sliding mat. The arrow indicates the direction of movement.

#### Preparation

- Use the straps to attach the Sliding Mat Wheelchair to the wheelchair backrest at the back of the seat. **Not on the sides!**
- Fold the Sliding Mat Wheelchair and place it on the seat.
- The sliding surfaces must face in towards one another.
- The tapered end with the pulling loops is routed through the gap between the backrest and seat.

#### Transfer

- The wheelchair user can be placed in a stable position with assistance by pulling on the loops.
- This process can be repeated several times.

Visit our website at [www.pm-med.de](http://www.pm-med.de) or watch our comprehensive product videos on our YouTube channel.

## 3.7 Approved accessories, combination with other products

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

## 4 Product marking / CE

The Petermann Sliding Mat Wheelchair is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, hereinafter called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

## 5 Cleaning, care and disinfection

Cleaning	Desinfection (if necessary)
60° washing machine	Recommended disinfectants for wipe disinfection: - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillol® 30 Foam - Bacillol® 30 Tissues - Bacillol® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Mild detergent	
Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently.	
Wash with similar colours only	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.
Recommended detergent for industrial washing machines: - - Maximo Liquid as single detergent	
Low dryer temperatures (45° - 50° C)	

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

- Mikrobac® Tissues\*
- Mikrobac® forte\*
- Bacillol® 30 Foam
- Bacillol® 30 Tissues

When using products marked with an \*, wipe down again using a water-soaked cloth after disinfection has been completed, in order to remove residues and prevent build-up. For wipe disinfection, please be sure to always follow the instructions provided by the manufacturer.

Always follow the care instructions on the label of every product as well as the information in the respective user manual.

## 6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-2070	Cotton/natural thickening, latex free	anthracite/red	200 kg
PM-2076	Cotton/natural thickening, latex free	anthracite/red	200 kg
PM-2078	Cotton/natural thickening, latex free	anthracite/red	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-2070	Sliding Mat Wheelchair	500 x 450 x 6	440 g	(01)04250355320704(10)x
PM-2076	Sliding Mat Wheelchair	1300 x 450 x 3	660 g	(01)04250355320766(10)x
PM-2078	Sliding Mat Wheelchair	1350 x 430 x 3	580 g	(01)04250355320780(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

## 7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

## 8 Disposal information

There are no specific disposal regulations for Sliding Mat Wheelchair, they can be disposed of in accordance with applicable national and local regulations.

**Petermann GmbH**  
**Waldweg 3**  
**D-91601 Dombühl**  
**Phone: +49 (0) 9868-9339-0**  
**Email: [petermann@pm-med.de](mailto:petermann@pm-med.de)**

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